Intended use

The UomoFit is a portable, non-sterile shoulder brace to be used for short-term immobilization and support of the shoulder while injured.

Indications

The UomoFit is used after:

- ✓ Bankhart repair;
- ✓ Capsular shift;
- ✓ Soft tissue repair / strain;
- ✓ Shoulder instability/injury.

The final decision concerning the usage and selection of the appliance shall be made by the healthcare professional.

Contraindications

⚠ UomoFit is not indicated for patients with pacemaker.

Fitting instructions

- ✓ Position the abduction cushion and the sling to your body. Place your forearm into the sling at a 90° angle. Place your thumb through the thumb loop. The thumb loop can be repositioned for improved fit and better comfort.
- ✓ Make sure that the magnetic buckles are closed around the strap of the abduction cushion.
- ✓ Close the strap of the abduction cushion around your waist.
- ✓ Place the adjustable strap around the neck.
- ✓ Bring the outside magnetic lock (farthest away from your body) to the buckle and it will lock automatically.
- ✓ Bring the innerside lock to the buckle to secure the strap.
- ✓ Use the single strap to fasten your forearm.
- ✓ When your doctor gave you permission to move your shoulder, you can open or close the abduction pillow with the zipper.

Care instructions


Used materials: Knitted fabrics out of cotton and/or polyamide and/or polyester

Cleaning: The UomoFit may be cleaned by hand using a mild soap and lukewarm water. Hang to dry.

⚠ Do not wring out. Do not iron.
Do not put the UomoFit in a washing machine or dryer.

**Warranty**

Currently valid legal conditions applicable.

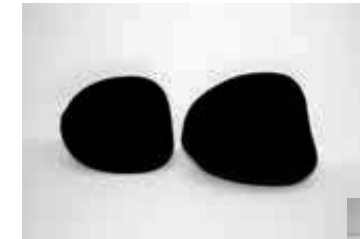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Important notes

- The brace is recommended for one user only.
- The brace is not intended to be applied on breached skin.
- Respect the cleaning instructions.
- Remove device before showering.

If allergic skin rashes occur, please consult your physician. Any serious incidents that occurred in relation to this device should be reported to the manufacturer and the competent authority of the member state in which the user is established.

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